

OBAGI®

SKINTRINSIQ™ DEVICE QUICK START REFERENCE GUIDE

SKINTRINSIQ™

INFEUSSE
INFEUSSE
INFEUSSE



Introducing the first professional-use device designed by Obagi. Powered by innovative technology, the SkintrinsiQ™ device extracts debris and impurities then infuses Obagi's proven skincare exactly where it's needed most. Accelerate patients' transformations by using the SkintrinsiQ system to take Obagi® skincare results to the next level.

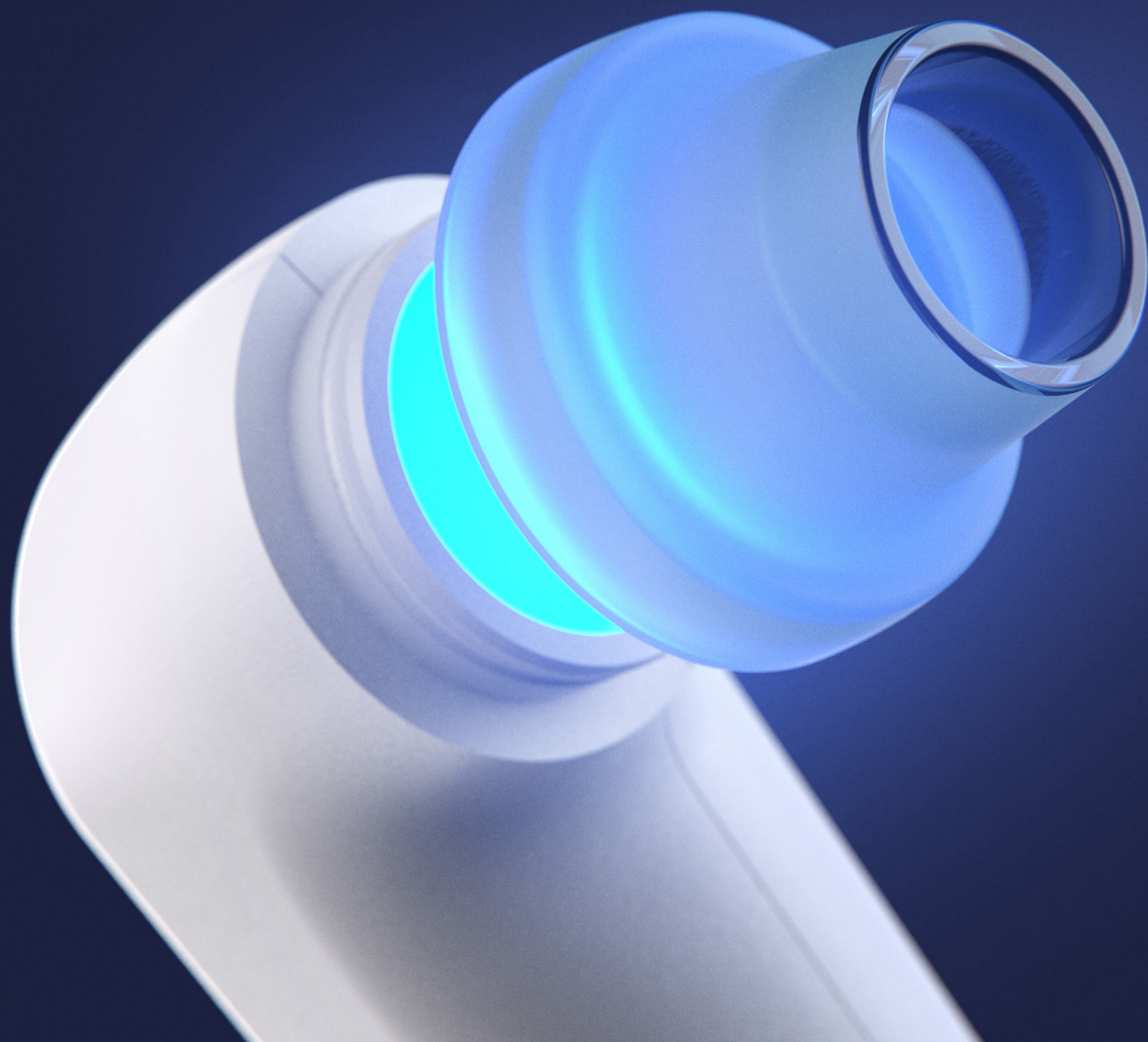


TABLE OF CONTENTS


How the SkintrinsiQ Device Works	04
Additional LED Benefits	05
Components Included in the SkintrinsiQ System	06
Obagi Protocol Kits	08
Obagi Treatment Protocol Overview	09
Step-by-Step Instructions	10
Patient Details	15



How the Skintrinsiq Device Works


The Skintrinsiq device uses pneumatic technology that provides negative and positive pressure at varying levels for different benefits throughout the treatment.

EXTRACT

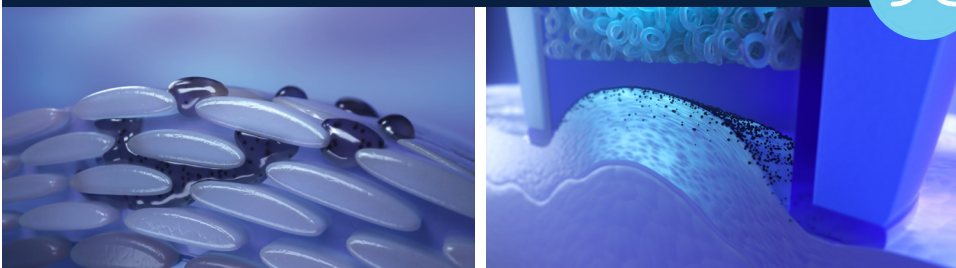


Apply the handpiece directly onto the skin. Then, choose if you would like to use either red or blue LED light therapy. Push the start button to begin extraction. As soon as the handpiece touches the skin, it pulls the skin upward and successfully extracts the debris due to a high level of pneumatic vacuum.

INFUSE

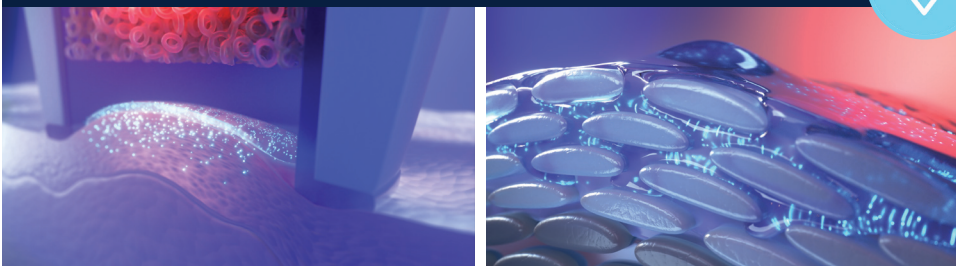

Apply Obagi product to face. Apply the handpiece directly onto the skin. Then, choose if you would like to use either red or blue LED light therapy. Push the start button to begin infusion. Please note: There is less of a vacuum pull on this step compared to extraction because product is being infused into the opened stratum corneum channels.

LOCK

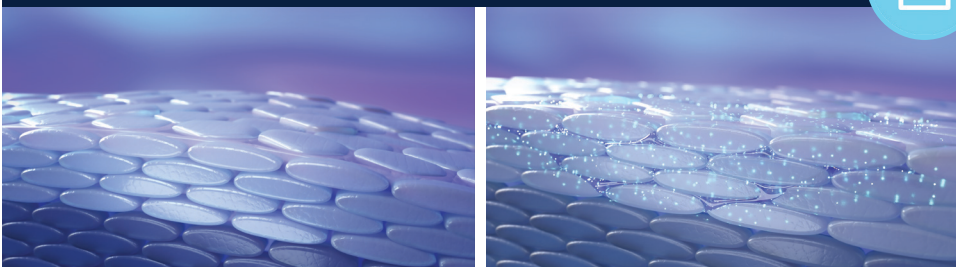


Once Infusion is complete, product remains in the skin to continue working after the treatment.

Additional LED Benefits

For this optional treatment, choose the best light for the patient's skin condition.

BLUE + RED LED LIGHT

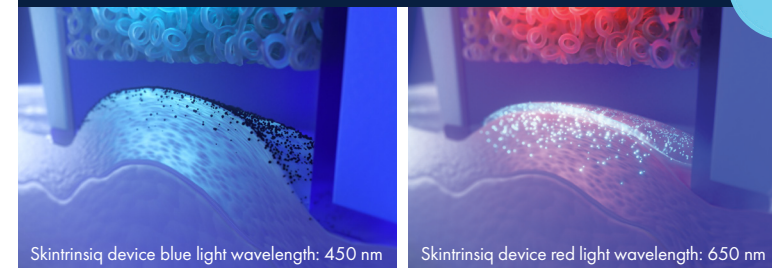

Studies have shown that blue LED light:

- Is suitable for use on acne prone skin.*
- Reduces surface bacteria.**
- Improves the appearance of oily, congested or clogged pores/skin.***

Studies have shown that red LED light:

- Promotes skin's natural renewal and improves signs of photodamage.†
- Helps skin look smoother and plumper.‡
- Soothes skin by reducing the appearance of superficial redness.§
- Finishes and locks benefits of Obagi Skintrinsiq device protocols.

TWO METHODS OF USAGE FOR HANDPIECE

Choose either glide or stamp, depending on administrator's preference and patient's needs:

- **Glide:** This method incorporates a smooth glide across the skin's surface. The glide method is ideal for extraction.
- **Stamp:** Place the handpiece down to form a seal against the skin. This initiates the LED and pneumatic pressure at the same time. Move the handpiece from one patch of the skin to another. The stamp method is ideal for infusion or problem spot/targeted areas.
- Please note: The light is on only while the vacuum is being applied. In the glide mode, the light is (pulsing) on continuously.

*Sorbellini, E., Rucco, M. & Rinaldi, F. Photodynamic and photobiological effects of light-emitting diode (LED) therapy in dermatological disease: an update. *Lasers Med Sci* 33, 1431–1439 (2018). **Shnitkind E., et al. *J Drugs Dermatol.* 2006;(7):605–610, ***Theravant. T-Delivery Document, †Li Y., et al. (2016) *PLoS ONE* 11 (6), ‡Barolet D., et al. *J Invest Dermatol.* 2009;129:2751–2759. Almeida Issa MC., et al. *Br J Dermatol.* 2009;161:647–653, §Barolet D. & Boucher A. *Lasers Surg Med.* 2008;40:106–112.

Components included in the Skintrinsiq system

One (1) Skintrinsiq Device

One (1) Handpiece and Umbilical Cord



ALSO INCLUDED:

One (1) Power Cord

One (1) User Guide

Obagi Protocol Kits (including Obagi Product & Tips) are sold separately.

Obagi Protocol Kits

All Skintrinsiq device treatments should be exclusively performed using Obagi products. Each Obagi protocol is designed to optimize the Skintrinsiq device's innovative technology: extracting impurities and infusing Obagi's proven products for visible patient results.

AMOUNT OF TREATMENTS PER KIT

Every protocol kit completes 30 treatments and includes 30 extraction tips and 30 infusion tips. Note: All treatment tips are single use.


KIT INFORMATION

How to use tips:

- Put on a new tip before the extraction phase of the treatment. Tips snap onto the handpiece. Press tip until you hear it click, which indicates it is locked into place.
- After phase is complete, remove the first tip by pulling it off the handpiece and discard.
- Put on a new tip before the infusion phase of the treatment.
- After phase is complete, remove the infusion tip and discard.

Reorder kits:

- Follow your normal Obagi reordering process by placing an order with your Obagi sales representative.

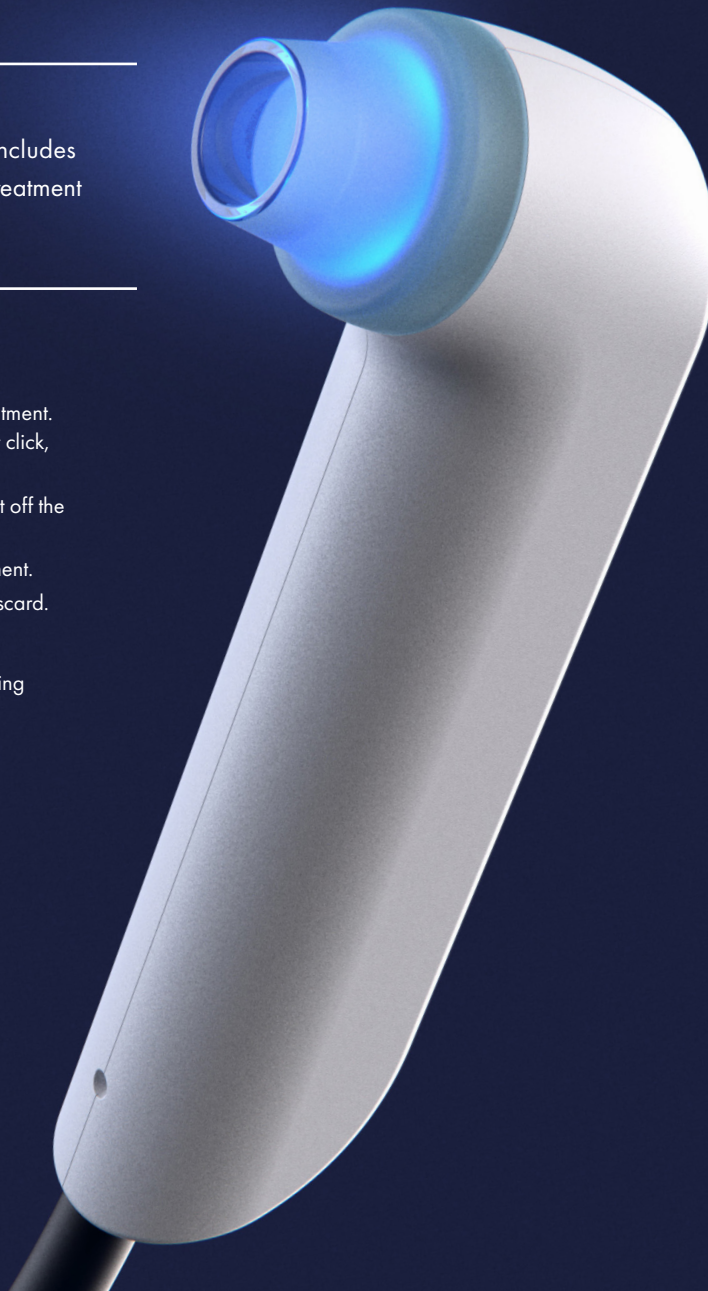

Obagi Treatment Protocol Overview*



Antioxidant:

Refresh and revitalize the appearance of skin

Obagi products included: Obagi Nu-Derm® Gentle Cleanser, Obagi Nu-Derm® Toner, Professional-C® Serum 10%, 15%, or 20%, Professional-C® Peptide Complex, Obagi Sun Shield Matte™ Broad Spectrum SPF 50 Sunscreen



Brightening:

Reveal a smoother, more radiant-looking complexion

Obagi products included: Obagi Nu-Derm® Gentle Cleanser, Obagi Nu-Derm® Toner, Obagi Nu-Derm® Clear Fx, Obagi Hydrate® Facial Moisturizer, Obagi Sun Shield Matte™ Broad Spectrum SPF 50 Sunscreen



Clarifying:

Improve oily, blemish-prone skin

Obagi products included: Obagi Nu-Derm® Gentle Cleanser, Obagi Nu-Derm® Toner, CLENZIderm M.D.® Pore Therapy^, Obagi Hydrate® Facial Moisturizer, Obagi Sun Shield Mineral™ Broad Spectrum SPF 50 Sunscreen



Firming:

Rejuvenate and replenish to minimize fine lines and wrinkles

Obagi products included: Obagi Nu-Derm® Gentle Cleanser, Obagi Nu-Derm® Toner, ELASTIderm® Facial Serum, Hydrate Luxe®, Obagi Sun Shield Mineral™ Broad Spectrum SPF 50 Sunscreen

*Follow all Obagi product guidelines


^CLENZIderm M.D.® Pore Therapy is a prescription item to be purchased per patient

Step-by-Step Instructions

POWER UP DEVICE



- Before first use, remove protective plastic film from screen.
 - Attach power cord .
 - Plug into power outlet.
 - Ensure handpiece is in its cradled position.
 - Turn on system using on/off button on back of the device.
 - Touch screen to start.

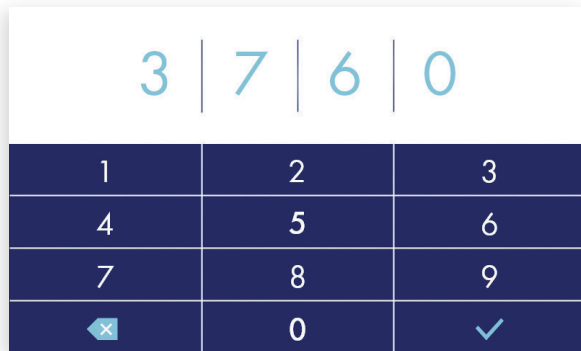


- Input User Code 3760. If code is incorrect, text will turn red.

SELECT TREATMENT




- Select Extraction.

EXTRACTION TREATMENT

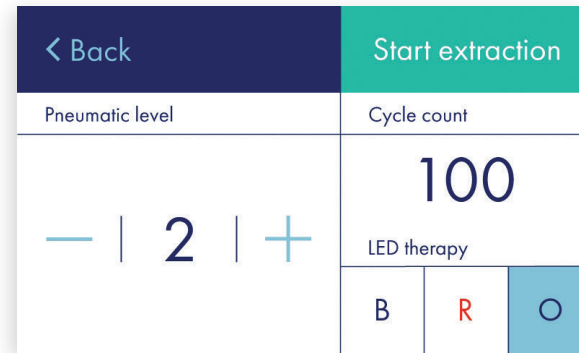

- Select pneumatic level and optional LED therapy with controls. Pneumatic level should be selected based on skin condition and thickness with 1 being the ideal level for thin, fragile skin; and 3 the ideal level for thick, oily and/or congested skin. The cycle count will change each time you pass the handpiece over the skin.

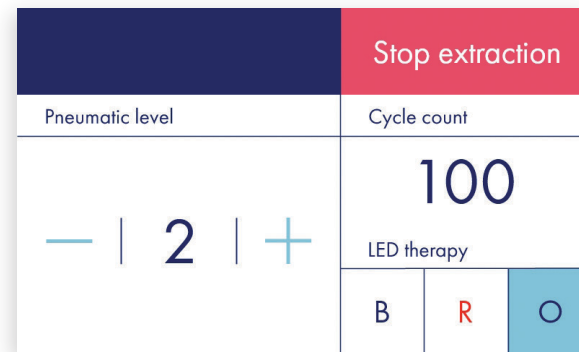

- Press Start Extraction.



- Click YES when the extraction phase is complete. Then, the device will automatically shut off.

Step-by-step Instructions


SELECT TREATMENT



7


Select Infusion.

INFUSION TREATMENT

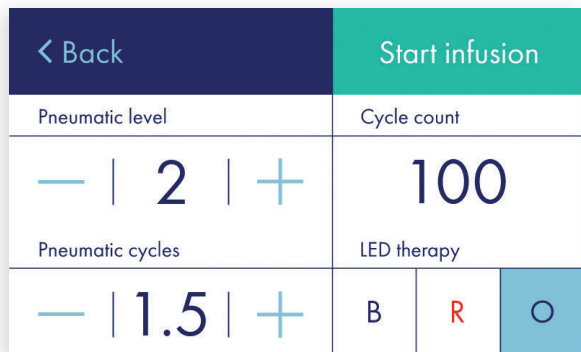

8

Select pneumatic level, pneumatic cycles, and optional LED therapy with controls. Pneumatic cycles affects the timing of the infusion with 0.5 being the shortest and 2.5 being the longest duration for the cycle. Cycle count will change each time you stamp the handpiece to the skin to infuse the product.

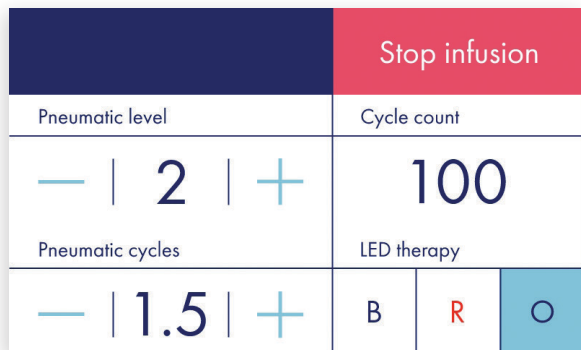

9

Press Start Infusion.

COMPLETE SKINTRINSIQ DEVICE TREATMENT



10

Click YES when the infusion phase is complete. This Skintrinsiq device treatment is now complete.

AT-HOME ROUTINE

Provide patients the suggested Obagi post-treatment at-home regimen, so patients can continue to experience Obagi product benefits.

POST-TREATMENT DEVICE MAINTENANCE

Wipe down the screen, handpiece, and device in between patient treatments with a damp cloth.

Tie up power cord around the system to store and prevent damage.

Dispose of all single-use extraction and infusion tips after each treatment.

Patient Details

PATIENT EXCLUSION CRITERIA

Patients with injured skin or any open lesions, any allergies to the products, anyone with atopic dermatitis, rosacea or a psoriasis flare, and anyone with a sunburn should not receive a Skintrinsiq treatment.

PRE-TREATMENT CONSIDERATIONS

Skintrinsiq treatments should only be performed on intact skin. Patients with cystic acne or moderate to severe inflammatory acne may require special consideration.

Some treatments should be discontinued for a period of time prior to a Skintrinsiq treatment. Following is the recommended amount of time a patient should wait between other treatments and a Skintrinsiq treatment.

- **5-7 DAYS:** Tretinoin, Salicylic Acid, Benzoyl Peroxide
- **5 DAYS:** Facial Waxing, Laser Hair Removal or Electrolysis
- **7 DAYS:** Dermal Fillers
- **14 DAYS:** Chemical Peel, Microneedling, Microblading
- **6 MONTHS:** Accutane, Chemotherapy, Radiation



All products/brand names, whether designated by notice or not (^{®/™}), are trademarks of Obagi Cosmeceuticals LLC and/or its affiliates. Distributed by Obagi Cosmeceuticals LLC.
©2021 Obagi Cosmeceuticals LLC. All rights reserved. 06/21 OBG.0043.USA.21.r1